

SimulEYE Membrane Peeling Kit (MPK)

INSTRUCTIONS FOR USE FOR THE EYE ITSELF:

1. *Open the reusable eye at the equator.*
2. *Place a retina inside the back half of the eye.*
3. *Snap the Eye front back onto the back half.*
4. *Ready to be used.*
5. *Attach the MPK eye to the Illuminated Base — press down at equator and not on cornea.*

6. *For added realism place some drops of water on the retina before closing. Work on peeling within 2-3 minutes max. after adding water to maintain membrane consistency. Water or Gel is not mandatory but can be used for added realism.*

ADDITIONAL INSTRUCTIONS FOR ACCESSORIES:

1. Base Tray — orient the Logos TOWARD THE SURGEON for stability.
2. Place Elevated Base in center of Tray or it can be placed off center for optimal hand support.
3. The 4 non-slip feet on the bottom of the Elevated Base just need to straddle the ridges in the Base Tray.